

# Custom Shape-Fitting Cushion

**LTV** SEAT™  
A ROHO GROUP PRODUCT





**You'll Love That Valve!**  
It lets you customize the cushion  
to fit your individual shape. ▶

Available in 3 colors and 2 fabrics.


Charcoal Gray Wool	Burgundy Ultraleather	Black Ultraleather
		

**GREAT FOR:**

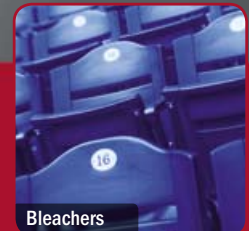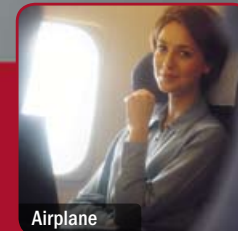

The secret to the extraordinary comfort of this multi-cell air cushion is the valve. It lets you customize the seat to your specific comfort level.

MEDICALLY PROVEN  
PRESSURE MAPPING  
ILLUSTRATES THE  
DIFFERENCE.

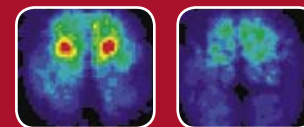

Red and yellow areas show painful pressure points. LTV significantly relieves pressure and maximizes comfort.



- Unsurpassed comfort
- Proven medical technology (DRY FLOATATION®)
- Reduces numbness and pain
- Completely portable – use it in the car, truck, airplane – anywhere
- Weighs only one (1) pound
- Easy-to-use hand pump included
- One-year limited warranty
- Available in black or burgundy ultra-leather and charcoal gray wool

**Size:**  
19" x 17" Height adjustable 1"-2".

**Package Contents:**  
This package contains the LTV zippered cushion with air insert and easy-to-use hand pump.

**LTV Cushion Instructions:**  
Adjusting the LTV Cushion

- Step 1** Unscrew the fill valve counterclockwise to open.
- Step 2** Add air using the pump until all of the cells are visibly filled. Screw fill valve clockwise to close, then remove the pump.
- Step 3** Place the LTV cushion on your chair with the non-skid side down and the zippered portion of the cushion at the back. The valve should be on the front left corner.
- Step 4** Sit on the LTV cushion in your normal sitting position.
- Step 5** Open the fill valve to adjust the air. Release air until you sink down into the cushion. Close the valve. The cushion is at its ideal inflation when there is about 1/2 inch of air cushioning between you and the chair. If you feel the chair through the cushion, add more air through the valve and adjust again.

**DO NOT OVER INFLATE**  
This product works most effectively with a minimal amount of air. You sink "into" the cushion.

**CAUTION**  
Protect the LTV cushion from pins or any other sharp objects as well as exposure to hot ashes.

**WARNING**  
According to the United States Coast Guard, the LTV cushion should NOT be used as a water flotation device.

MADE IN THE U.S.A.



**shape fitting technology, inc.**

100 north florida avenue  
belleville, il 62221-5429 USA  
call toll-free 800-851-3449  
outside U.S. and Canada  
call 618-277-9150  
[www.shapefitting.com](http://www.shapefitting.com)

Exclusive distributor of ROHO® DRY FLOATATION® Technology in global consumer markets

DRY FLOATATION®, AIRHAWK™, and LTV™ are registered trademarks of The ROHO Group. Sleepmatterzzz® is a registered trademark of Sleepmatters, Inc.  
© 2004 The ROHO Group, Inc.

We've solved the world's most difficult seating problems. Imagine what we can do for you.

Cair-pillow



The Cair-pillow provides comfort, therapy and support – home or away! Adjustable, shape-matching air pillow is also available with hot/cold gel pack. Quickly deflates for easy travel.

Select Series Cushions



The latest innovation in wheelchair cushions. New patented technology allows wheelchair users to easily change positions as often as needed with a simple push of a knob.

Motorcycle Comfort Seating System



AIRHAWK absorbs the shock and vibrations of the engine and road to make every mile of a motorcycle ride as comfortable as the first.

Truck Comfort Seating System



Truck drivers have a soothing ride, without the numbness and backaches that come with hours of driving.